

Instruction for use

Description

VivaCol is a specialized skin barrier specifically designed for two primary applications: stoma care and gap filling during Negative Pressure Wound Therapy (NPWT). This product is particularly beneficial for use in challenging anatomical areas, such as the sacrum, where traditional wound care solutions may struggle to provide adequate support. Formulated with carboxymethylcellulose (CMC), VivaCol offers a unique combination of properties that enhance wound management. CMC is known for its excellent moisture-retentive capabilities, which help maintain an optimal healing environment by preventing excessive dryness and promoting tissue hydration. This is crucial for effective wound healing, especially in delicate or irregularly shaped areas.

In stoma care, VivaCol acts as a protective barrier, shielding the skin from irritation and damage while ensuring a secure fit around the stoma.

When used in conjunction with NPWT, it fills gaps and irregularities in the wound bed, providing a stable foundation for the therapy to work effectively.

Overall, VivaCol is engineered to deliver reliable support and protection in wound management, making it an essential tool for healthcare providers aiming to optimize patient outcomes in complex wound care scenarios.

Application Instructions:

1. Clean and thoroughly dry the skin area around the stoma or the gap.
2. Open the tube by removing the cap, squeeze out the VivaCol paste, and apply it directly onto the skin around the stoma or in the gap.
3. As a gap filler, apply a sufficient amount of VivaCol to eliminate the gap.
- 4- As a skin barrier, Shape the ostomy skin protective ring to fit the skin around the stoma.
4. If needed, gently smooth the leak-proof ointment using a wet knife or finger.
5. Place the NPWT dressing or adhesive PU-film or ostomy skin barrier over the stoma, pressing gently with the paste to secure it in position.
6. After use, clean any remaining paste from the tube exterior and reseal the cap to prevent moisture.
7. Use warm water and a towel to remove the paste from the skin. Some oily residue may remain but will not affect the new skin barrier's performance.

Caution:

1. Discontinue use and seek advice from a healthcare professional if irritation occurs.
2. Avoid applying the paste over the entire adhesive area of the skin barrier.

Contraindication:

Do not use on individuals with a known sensitivity to CMC.

Storage:

VivaCol should be stored in a cool, dry room when not in use.



Protect against direct sun light.



Keep dry.